

Söring

short clinical abstract:

Special field: Gynaecology

Sub: Laparoscopy

Indication: Adhaesiolysis

Doctor in charge: Dr. JOHAN VAN DER WAT,
Park Lane Clinic, Johannesburg, South Africa

Conventional Treatment: Normally blunt or sharp dissections are satisfactory, but often, significant bleeding can occur.

Using of the Arco™ : Argon-coagulation-setting 3 (50W; 0,5 l/min). The Arco™ is effective for accomplishing haemostasis. For good success the probe must merely passing the bleeding tissue.

Advantage of the Arco™ : The Arco™ is especially effective in oozing areas when actual vessels are difficult to identify and control. The low gas range flow is very safe. Bleeding can be controlled without smoke production. The depth of tissue coagulation is limited to 2-3 mm.

Used handpieces: 2 mm flexible probe, 40 mm long

Literature: Own documents, Video: Gynaecology