

# Söring

**short clinical abstract:**

**Special field: Gynaecology**

**Sub:** Laparoscopy

**Indication:** Ectopic pregnancy  
(Haemostasis)

**Doctor in charge:** Dr. JOHAN VAN DER WAT,  
Park Lane Clinic, Johannesburg, South Africa

**Conventional Treatment:** Laparotomy or haemostasis with conventional monopolar or bipolar coagulation.

**Diagnosis:** Fallopian pregnancy

**Use of the Arco™ :** Argon-coagulation-setting 3 (50W; 1,5 l/min).

**Advantage of the Arco™ :** The blood on the coagulation site is blown away with the stream of argon gas. The absence of smoke is important. Because of the non-touch technique, eschar is not removed inadvertently. The speed of tissue coagulation is faster than with conventional techniques.

**Used handpieces:** 5 mm straight 30°C probe , 40mm long

---

**Literature:** Own documents,