

# Söring

**short clinical abstract:**

**Special field: Gynaecology**

**Sub: Laparoscopy**

**Indication: Myomectomy**

**Doctor in charge:** Dr. JOHAN VAN DER WAT,  
Park Lane Clinic, Johannesburg, South Africa

**Conventional Treatment:** Abdominal myomectomies are performed annually to conservatively treat symptomatic uterine Leiomyomata. The treatment of choice has been hysterectomy.

**Use of the Arco™ :** Argon-coagulation-setting 4 (65W; 2 l/min). Direct application by small myoma and coagulation the myomabed after removal.

**Advantage of the Arco™ :** Argon beam coagulation, which delivers conventional monopolar coagulation current in a non-touch technique, is considered to be a safe and effective tool for laparoscopic removal of symptomatic leiomyomata. The Arco™ allows easier dissection and rapid haemostasis during operation, resulting in both reduced blood loss and operating time. It is useful in achieving haemostasis.

**Used handpieces:** 5 mm straight probe, 40 mm long

---

**Literature:** Own documents, Video: Gynaecology