

# Söring

**short clinical abstract:**

**Special field: Gynaecology**

**Sub:** Laparoscopy

**Indication:** Second infertility  
(Parovariancyst)

**Doctor in charge:** DR. JOHAN VAN DER WAT,  
Park Lane Clinic, Johannesburg, South Africa

**Conventional Treatment:** Epophorectomy.  
Lasers of various types were used for laparoscopic treatment of endometriosis. Now there is a return to the electrosurgical method for most of the laparoscopic endometriosis treatment.

**Diagnosis:** Suspicion of tubocornual obstruction. The laparoscopic diagnosis shows normal tubes and ovarias but embryonic remnants at the parovarium.

**Use of the Arco™ :** Argon-coagulation-setting 3 (50W; 0,5 l/min). Coagulation of the epophoroncysts by burning a hole inside and dry them out.

**Advantage of the Arco™ :** Because of the low gas range flow there is no risk for gasembolie.

**Used handpieces:** 2 mm straight probe, 40 mm long

---

**Literatur:** Own documents, Video: Gynacology