

Translation of Operation-Report form 22.10.02

OP – Report regarding Mamma – Ablatio / St. Joseph – Stift Bremen/Germany!

User: Senior Physician Mr Dr. Parys / Gynecology St. Joseph – Stift Bremen
Senior Consultant Mrs. Prof. Frantzen!

Operation: Mamma – Ablatio!

The skin cut was carried out with pure Cut-current set to 130 watts. The coagulation setting was 55 watts and was only needed for hemostases directly on the skin.

For the whole preparation of the breast / fat tissue only Blend Cut current was used.

Adjustment: Blend Cut 120 watts / sub function = bL. 8!

According to the user the cut and coagulation results were so convincing, that a change to the forced cutting (using Force coagulation for cutting purposes) / Sub function = Fr. 9 was not requested

A bipolar use was not requested by the user

The suggestion, that an extremely well cutting and coagulation potential is implemented in the Söring devices was clearly confirmed, in Cut as well as in Coag mode.



Einstellwerte



Bild 1



Bild 2



Bild 3



Bild 4



Bild 5